

Read the [Introduction](#) for more information on these standards, including where to direct comments, questions, and recommendations. As new items are introduced, current items are discontinued, and/or health and safety issues arise, these standards will be revised to provide updated information. Sort by Update Date to view recent changes.

## **Litter – S.K.E.D**

NFES Status

Active

NFES #

001670

Category

Medical

Updated

Mon, 05/01/2017 - 12:00

Storage and Shelf Life Checks

None

## **Initial Inspection/Disposal Criteria**

1. Inspect for blood or other bodily fluids, alert supervisor for further instructions.
2. Visually inspect for cuts or tears in plastic surface, soiled surface, missing parts such as straps or fasteners. Inspect for structural damage such as grommets pulled out. Inspect all straps; handles (web gear) are intact and functional.
3. Return to service if item is clean and in unused condition.
4. Refurbish if item is soiled or has damaged components that are easily repaired or replaced.
5. Dispose of S.K.E.D. if there are cuts or tears to plastic materials, webbing and straps are damaged beyond repairs that is economically feasible. See local protocol for proper disposal of items that possess biological hazards.
6. Retain all serviceable components for replacement on other litters.

## **Refurbishing Procedures**

### **A. Cleaning**

1. Wash S.K.E.D. with a disinfectant and warm water.

2. Wash with power washer and hang to dry.

## **B. Repair**

1. Replace worn or damaged straps or fasteners.
2. Replace missing or damaged grommets.

## **C. Testing for Performance**

- Reassemble to ensure completeness and all parts are fitting properly.

## **D. Repackaging**

1. Roll up S.K.E.D. (using rubber gloves for better grip) small enough to fit into case.
2. Fasten retaining strap tightly so S.K.E.D. can be easily extracted.

### Reference

For information and parts lists see:

[SKEDCO, Inc.](#)

PO Box 230487

Portland, OR. 97281

Phone: 800-770-7533